



91 Carolyn Blvd. Farmingdale, NY 11735 800-321-1100 • www.trisupreme.com



AR coated demo lenses, hard case and micro-fiber cleaning cloth included.





#### Let your lab stock your frame chassis for you.





















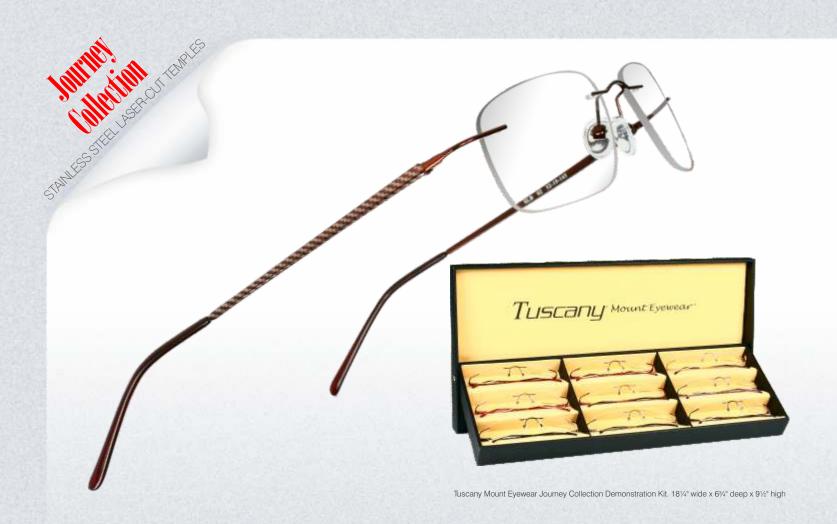


AR coated demo lenses, hard case and micro-fiber cleaning cloth included.



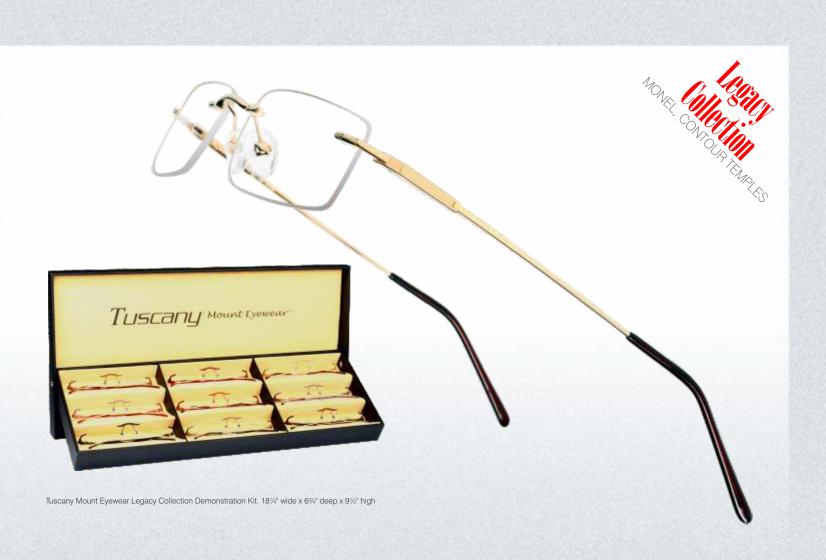


#### Let your lab stock your frame chassis for you.



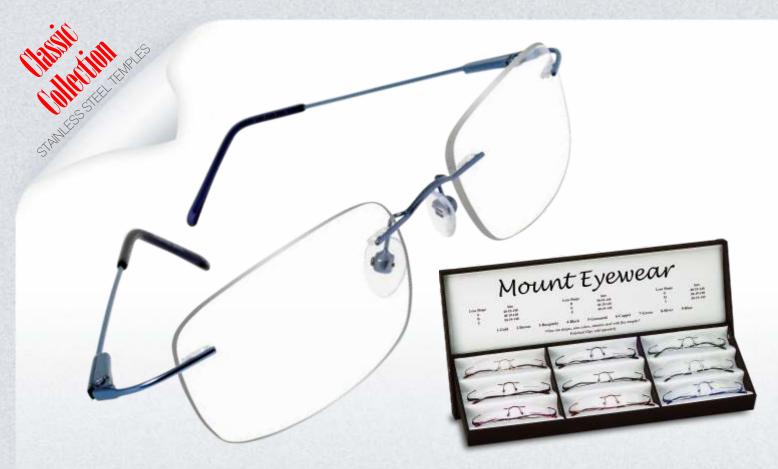

Tuscany Mount Eyewear Classic Collection Demonstration Kit. 181/4" wide x 63/4" deep x 91/2" high

Hard case and micro-fiber cleaning cloth included. Clip sold separately.

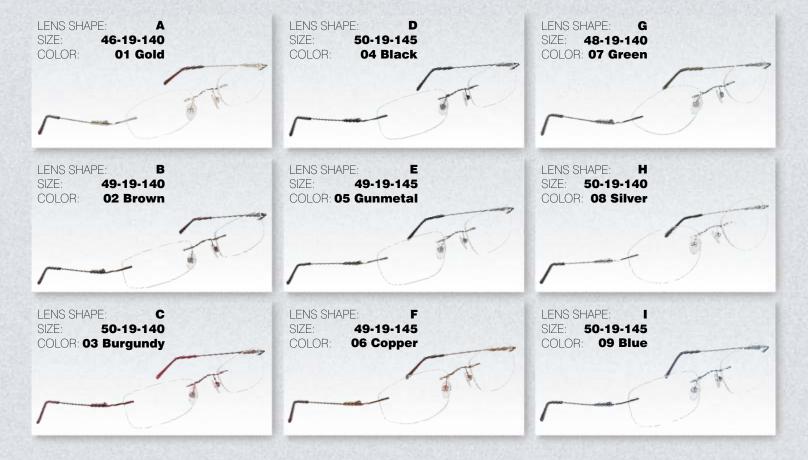


### TUSCANY® Mount Eyewear™


## TOOLS REQUIRED COMPRESSING/BRACING PLIER FLUSH CUTTING PLIER # 1027 **TUBE REFRESHER** # 2085 LENS PATTERN **EXAMPLE ONLY DRILL CHART EXAMPLE ONLY** ALL MOUNT EYEWEAR™ DRILL HOLES ARE Ø 1.4 MM

#### **ASSEMBLY INSTRUCTIONS**


Edge the lenses using the supplied **Pattern**. Then drill the lenses according to the **Drill Chart**. All Mount Eyewear™ drill holes are Ø 1.4 mm. We recommend using polycarbonate or Trivex® lenses.

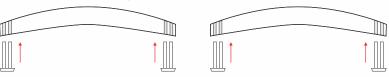


Push the plastic compression mounts into the lenses from the back. The compression mounts must be a snug fit in the drill holes.

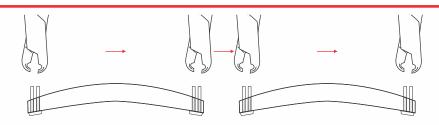

Clip off the excess plastic of the compression mount flush to the lens surface with the **Flush Cutting Plier**.

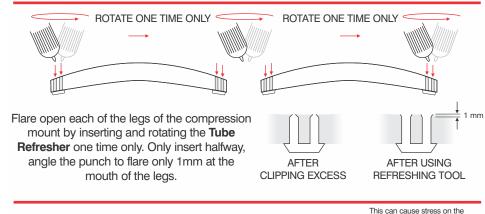

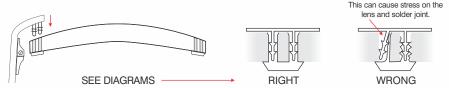

Carefully insert a temple into legs of the compression mount. Ensure that the barbed legs of the temple are inserted evenly into compression mount and the plastic is not distorted.


Do NOT bend the legs.

Incorrect assembly at this point will shorten the life of the frame.



Clamp tightly with the

Compressing/Bracing Plier. Repeat with
the other temple. Finally, assemble the
bridge using the same procedure.

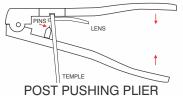

# 5021

Mounts can be disassembled with this special tool. Insert the frame and lens into

the tool. Center the two pins on the tool over the mounts and close down.

## Tuscany Mount Eyewear

**Studio Collection** 

**Signature Collection** 


#### Lens Lens Shape Shape OTA PMA Lens Lens Shape **OTB** Shape **PMB** Lens Lens Shape Shape OTC **PMC** Lens Lens Shape Shape OTD PMD Lens Lens Shape Shape OTE **PME** Lens Lens Shape Shape OTF PMF Lens Lens Shape Shape **PMG** OTG Lens Lens Shape Shape OTH РМН Lens Lens Shape Shape OTI PMI

## Tuscany Mount Eyewear


#### **Journey Collection Legacy Collection** Lens Lens Shape Shape SLA MUA Lens Lens Shape Shape MUB Lens Lens Shape Shape SLC MUC Lens Lens Shape Shape MUD Lens Lens Shape Shape MUE Lens Lens Shape Shape MUF Lens Lens Shape Shape MUG Lens Lens Shape Shape SLH MUH Lens Lens Shape Shape SLI MUI

## Tuscany Mount Eyewear

#### **Classic Collection**

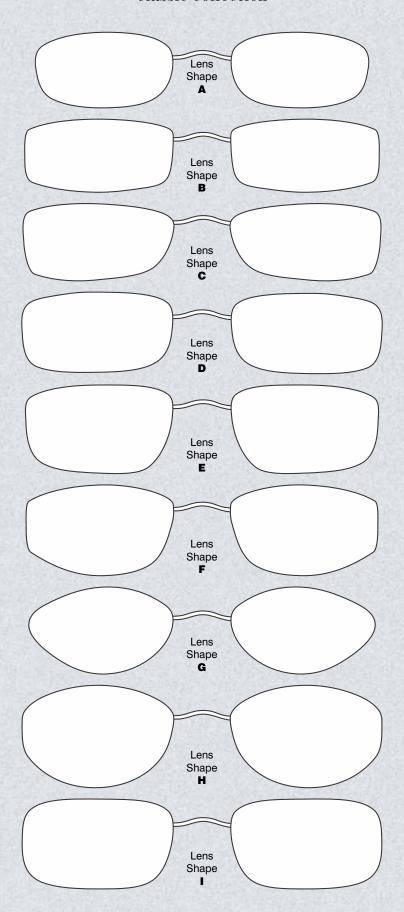



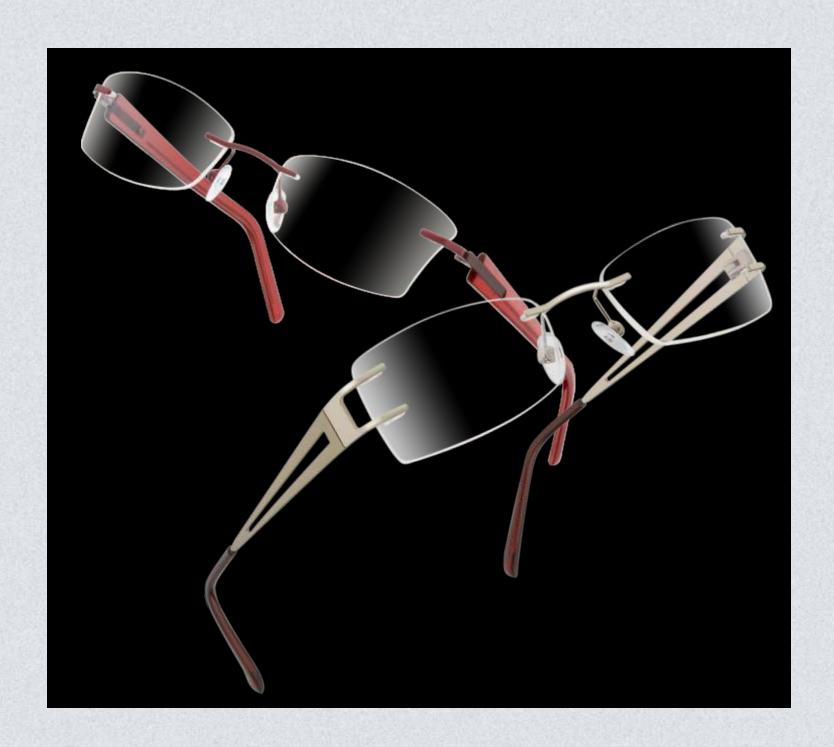

# Visibly Different Tri-Supreme-Hirsch OPTICAL

91 Carolyn Blvd. Farmingdale, NY 11735 800-321-1100 • www.trisupreme.com